




Step by step Upload Image From CD/DVD into PACS

1. Open PACS UV application.
2. Insert CD/DVD into CD/DVD drive.
3. Click import from CD

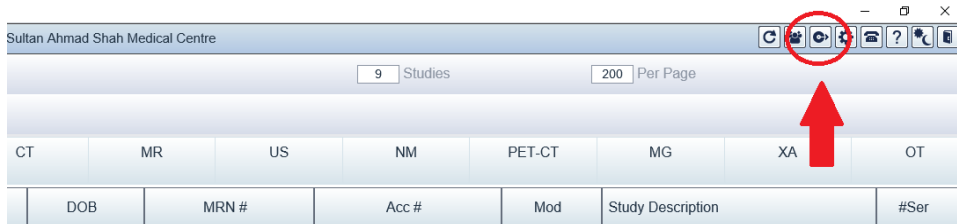

4. Select CD/DVD

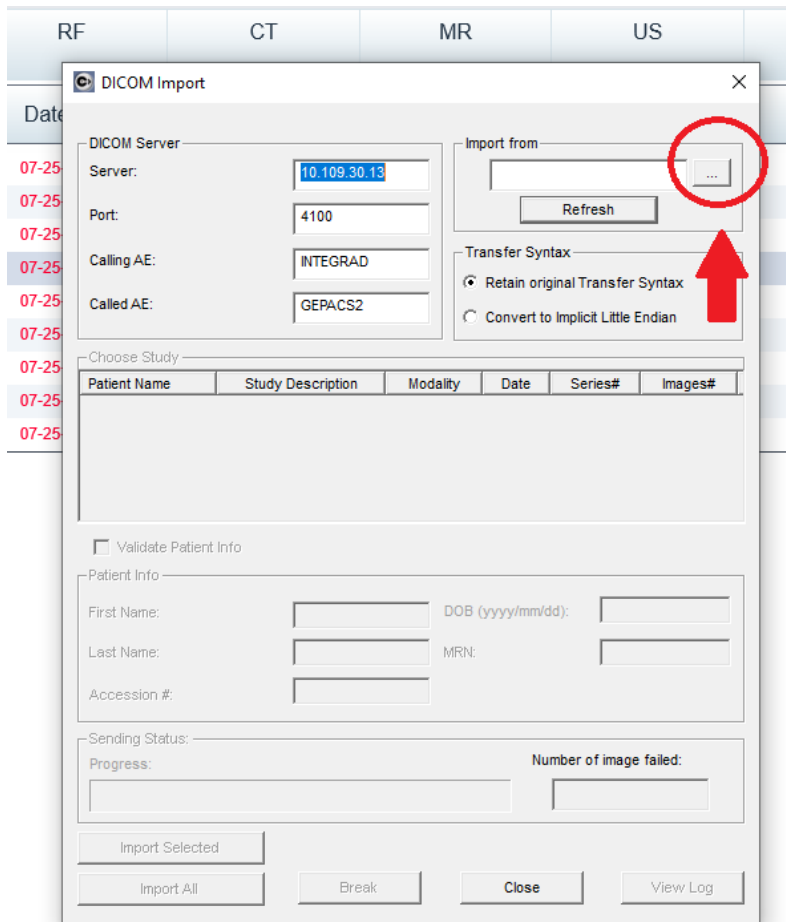

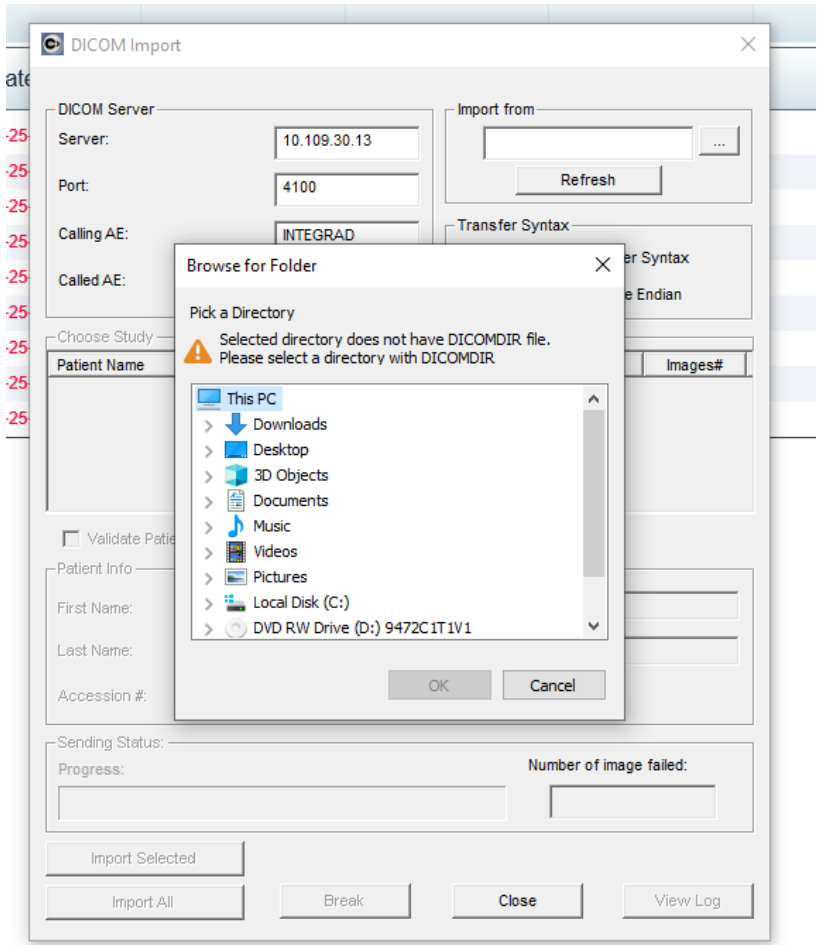

5. Click Import All

DICOM Import

DICOM Server

Server: 10.109.30.13

Port: 4100

Calling AE: INTEGRAD

Called AE: GEPACS2

Import from: D:\

Refresh

Transfer Syntax

Retain original Transfer Syntax

Convert to Implicit Little Endian

Choose Study

Patient Name	Study Description	Modality	Date	Series#	Images#
MAT RAFI BIN JU...	Xray Chest, 1 view	CR	20220...	1	1
MAT RAFI BIN JU...	CT THORAX - PLAIN	CT	20220...	8	785

Validate Patient Info

Patient Info

First Name: DOB (yyyy/mm/dd):

Last Name: MRN:

Accession #:


Sending Status

Progress: Number of image failed:


Import Selected

Import All Break Close View Log


6. Wait until sent DICOM is completed

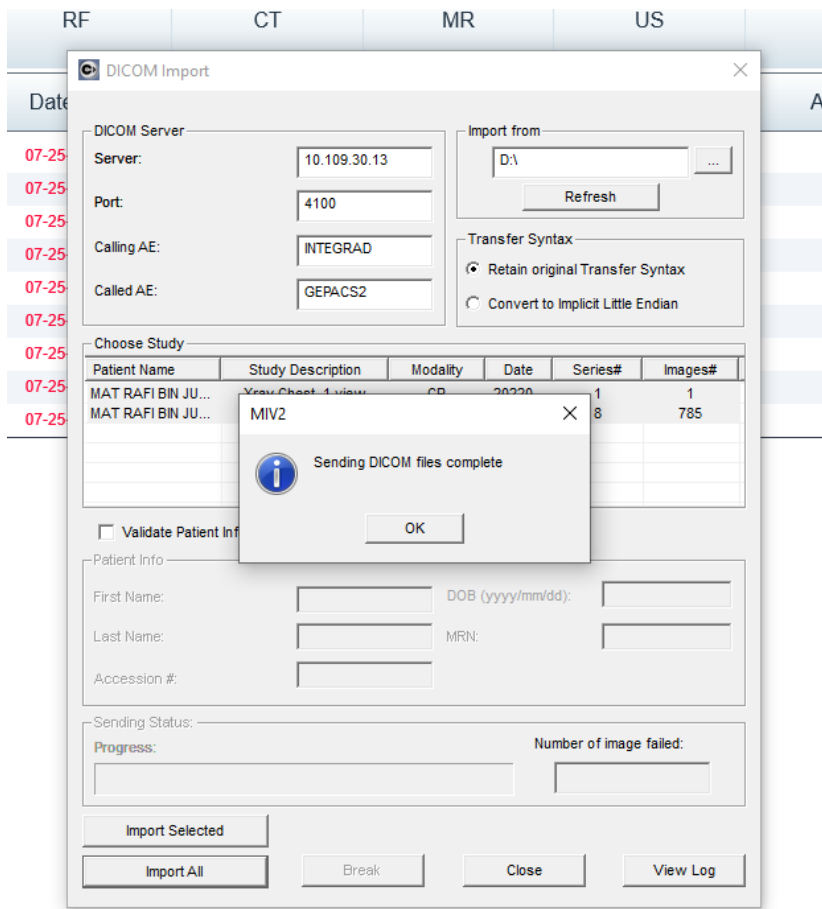


7. Register Patient in RIS & Start the exam

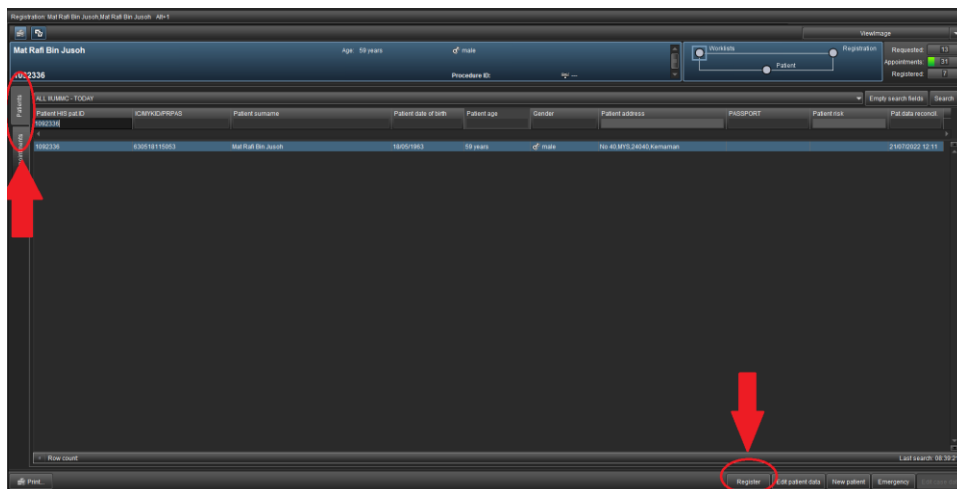

8. Open CPACS & Select Centricity EM

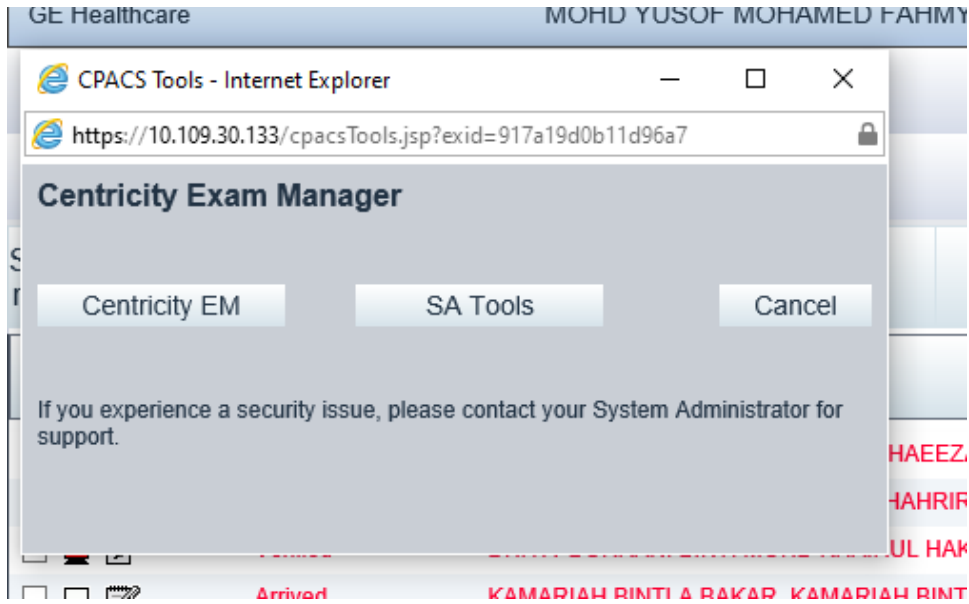

9. Select Reallocated & Merge

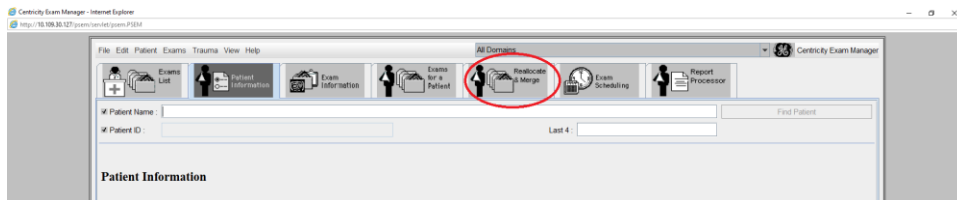

10. Merge patient information first (if patient information is different with RN SASMEC)

File Edit Patient Exams Trauma View Help Centricity Exam Manager

Exam List Patient Information Exam Information Exams for a Patient Reschedule & Merge Exam Scheduling Report Processor

Patient Merge Mode

From Source Patient:

Patient Name: MAT, RAFI BIN Birth Date: 1963.05.18 Gender: M

Patient ID: 300628 Last: 0628

Accession Number	Modality	Procedure	Sch. Date & Time	Status	Img. Count
	CR	Unspecified CR CHEST	2022.07.11.22.09.25	Arrived	1

To Destination Patient:

Patient Name: F1 BIN JUSOH, MAT RAFI BIN JUSOH Birth Date: 1963.05.18 Gender: M

Patient ID: 1092336 Last: 2336

Accession Number	Modality	Procedure	Sch. Date & Time	Status	Img. Count
184523	DX	CHEST AP	2022.07.13.20.37.10	Verified	1
180633	CR	X-RAY UPLOADED IMAGES	2022.07.25.08.42.27	Ordered	0
180632	CT	CT SCAN UPLOADED IMAGES	2022.07.25.08.42.27	Ordered	0

11. Merge image

Exam Merge Mode

From Source Exam: Show Series

Patient Name	Patient ID	Accession Num.	Order No.	Modality	Procedure	Sch. Date & Time	Status	Img
AT RAFI BIN JUSOH	1092336							
MAT RAFI BIN JU.	1092336	184523	184523	DX	CHEST AP	2022.07.13.20.37.10	Verified	1
MAT RAFI BIN JU.	1092336		1155990/27	CR	Unspecified CR CHEST	2022.07.11.22.09.25	Arrived	1
MAT RAFI BIN JU.	1092336	180633	180633	CR	X-RAY UPLOADED IMAG.	2022.07.25.08.42.27	Ordered	0
MAT RAFI BIN JU.	1092336	180632	180632	CT	CT SCAN UPLOADED IM.	2022.07.25.08.42.27	Ordered	0

To Destination Exam: Show Series

Patient Name	Patient ID	Accession Num.	Order No.	Modality	Procedure	Sch. Date & Time	Status	Img
MAT RAFI BIN JU.	1092336	184523	184523	DX	CHEST AP	2022.07.13.20.37.10	Verified	1
MAT RAFI BIN JU.	1092336		1155990/27	CR	Unspecified CR CHEST	2022.07.11.22.09.25	Arrived	1
MAT RAFI BIN JU.	1092336	180633	180633	CR	X-RAY UPLOADED IMAG.	2022.07.25.08.42.27	Ordered	0
MAT RAFI BIN JU.	1092336	180632	180632	CT	CT SCAN UPLOADED IM.	2022.07.25.08.42.27	Ordered	0

Match Unspecified Exams

12. Finish EXAM